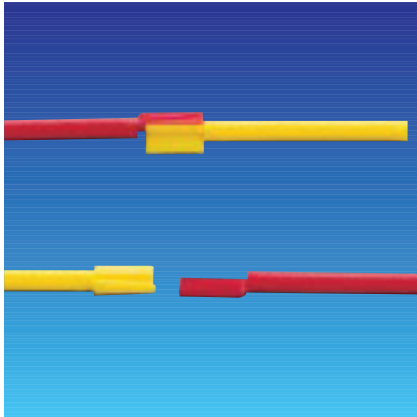
# CPA-Geschiebe / CPA-Attachment

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*The CPA is an extra coronal inactive slide connection attachment. Male and female both have integrated paralleling tools which makes application easy. The conical CPA can be used for fixed bridges that do not have parallel preps or for removable structures combined with a special milling.*

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
### Advantages:

- Inexpensive
- Can be adapted in two directions
- Both primary and secondary structure can be cast at the same time
- No rotation due to conical shape.

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
**Order No.: 3114** CPA-Set, containing 6 males and 6 females  
**Order No.: 3114K** 1 Ceramic spacer

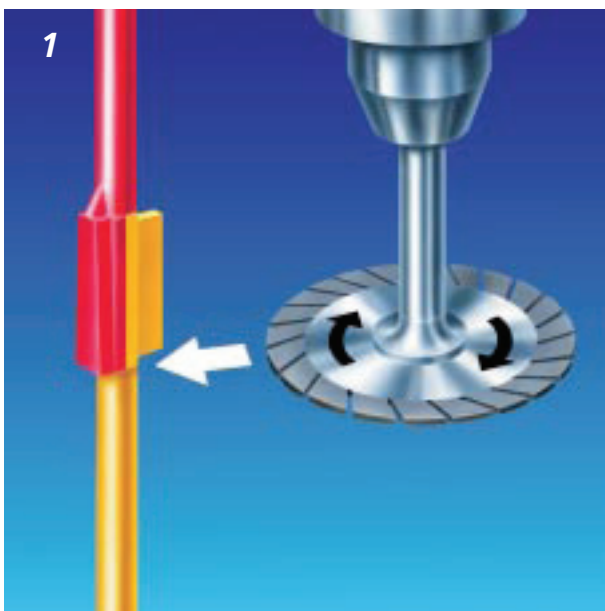**Fig. 1:**

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*Separating the paralleling tool from the male.*

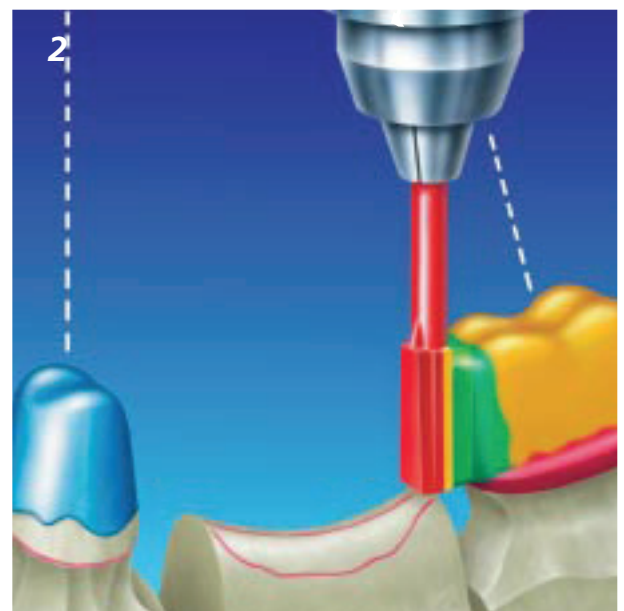
**Fig. 2:**

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*Attaching the male using the female as a paralleling tool in the insertion path of the opposite abutment tooth.*

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
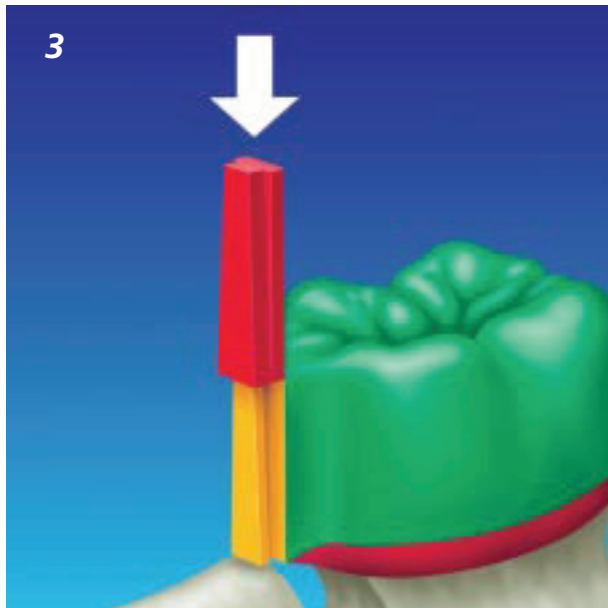
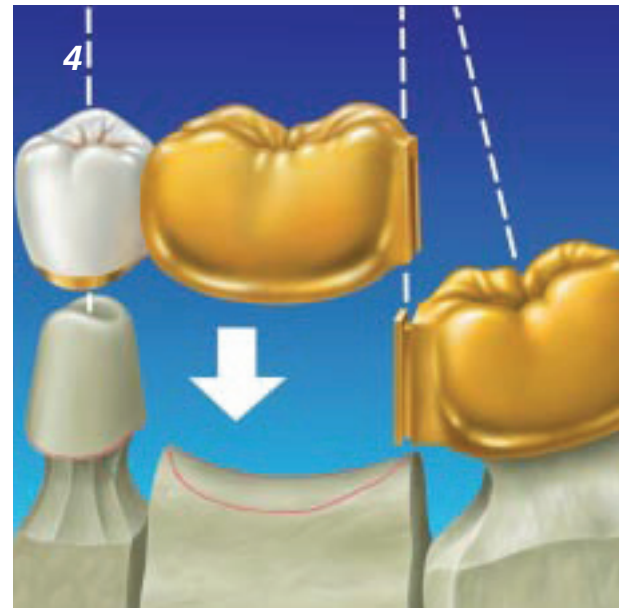
*After the secondary casting is finished and polished it can be easily attached to the primary construction, even in the case of non-parallel die position.*

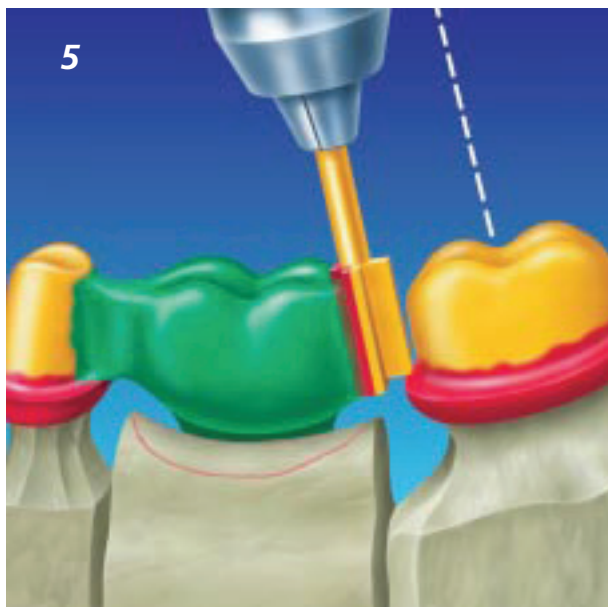**Fig. 5:**

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*Attaching the female using the male as a paralleling tool in the insertion path of the molar tooth.*

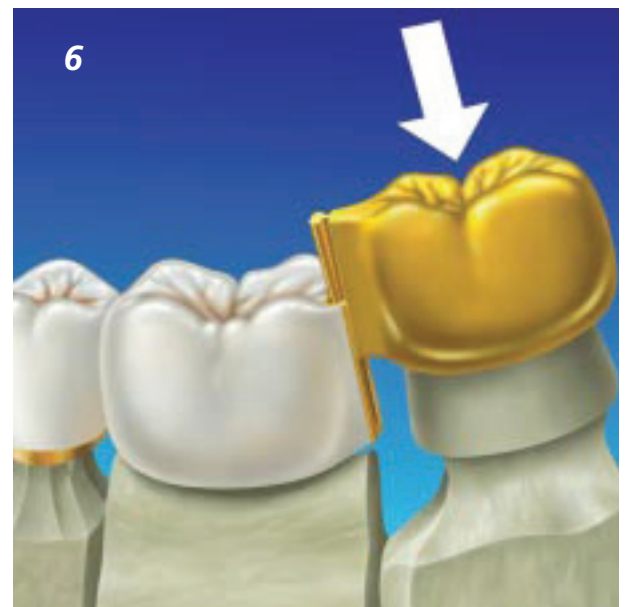
**Fig. 6:**

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*After the secondary casting is finished and polished it can be easily attached to the primary construction, even when the dies are not parallel.*

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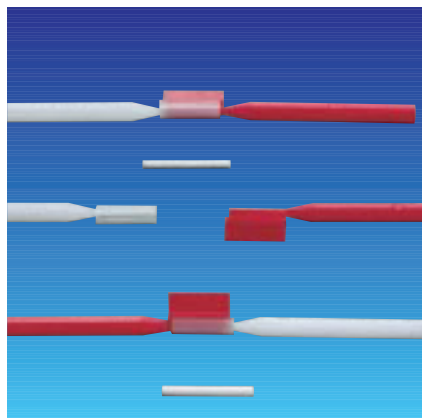*The OM is a small intra coronal inactive cylindrical slide attachment made of burn out plastic. It is closed to the gingival area. Male and female both have integrated paralleling tools which makes application easy. The OM can be used for fixed bridges that do not have parallel preps or for removable structures combined with a special milling especially in the anterior area due to its small dimensions.*

#### Advantages:

- Inexpensive
- Can be adapted in two directions
- Both primary and secondary structure can be cast at the same time
- Can be used in the anterior region.

**Order No.: 3020**     $\varnothing$  1.3 mm  
Contains 6 males and 6 females,  
incl. 6 ceramic pins

**Order No.: 3020K**    Ceramic pin for OM  $\varnothing$  1.3 mm,  
contains 6 pieces

**Order No.: 3021**     $\varnothing$  1.0 mm  
Contains 6 males and 6 females,  
incl. 6 ceramic pins

**Order No.: 3021K**    Ceramic pin for OM  $\varnothing$  1.0 mm,  
contains 6 pieces