BAR-Connector-System

Indikation:

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Indication:

The BAR-Connector-System consists of differently profiled bars made of burn out plastic, spacers and females made of a highly elastic and precise acrylic material. It has a wide application range and is often used in the situation as shown.

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Fig. 1:

Choose the most suitable bar for the existing situation.

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Fig. 2:

Adapt the profile to the jaw ridge taking care of hygienic aspects. Then invest and cast as usual.

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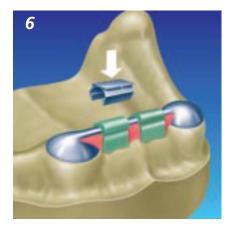
Fig. 5, 6:

Place spacers over the bar adapting to the gingiva and fill out the undercuts. Now finish the denture.

The spacers do not bond to the resin and can be removed easily. It is also possible to use a metal housing over the spacers.

Fig. 7:

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After completely finishing the denture remove the spacers and, using the insertion tool, position the females into the provided space. There is a choice of different females. Yellow would be for a rigid and orange for a resilient situation. There are three females with reduced inside diameter to choose from, if there should be a loss of friction.

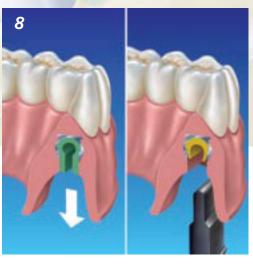
Fig. 8:

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In this case a metal housing was used. Process as shown in fig. 6.

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